

## Product Description

The 3ET EvacSHEET is designed to evacuate bedbound patients on their hospital bed mattress.

The EvacSHEET system is quick and simple to use and by utilising the mattress under the

patient, it offers the highest degree of comfort and protection in an emergency evacuation.

The EvacSHEET is available in 3 variants: Standard, Paediatric and Trolley. Each of the EvacSHEET variants has "anti-capsize" stability, 3 x colour coded securing cross straps to accommodate different patient sizes & conditions as well as good ergonomic design features to ensure the persons performing the evacuation are not put at risk of moving & handling injuries.

The Standard and Paediatric EvacSHEET's are permanently fitted to the underside of the hospital

bed mattress with all the securing straps and pulling handles stored away in the EvacSHEET pockets and secured with poppers, so once fitted there is no visual signs of the emergency evacuation system being there. The Trolley EvacSHEET is only fitted when required to evacuate in an emergency.

The 3ET EvacSHEET can be operated safely by 2 persons, however depending on the size of the person to be evacuated and the size of the people available to perform the evacuation, the EvacSHEET can accommodate up to 8 operatives utilising the additional pulling loops fitted to the main pulling straps.

Features

- Heavy-duty Flame-Retardant material with wipe clean finish.

- · Easily assessable, secure pockets for storage of all cross straps and pulling straps.
- Heavy duty elastic corner straps for attaching securely to underside of mattress.
- Ergonomically designed Hi Visibility "soft touch" pulling handles.
- 3 x colour coded cross straps fitted with heavy duty plastic trident buckles.
- · Unique "anti-capsize" design for extra stability and security.
- $\cdot$  Can be used on AirFlow mattress systems.
- Extra pulling loops to accommodate up to 8 operatives
- Individual product serial number and washing instructions.
- Machine washable @ 70 degrees for infection control.
- $\cdot$  EvacSHEET can be easily tilted to negotiate narrow doors or openings and other obstacles.
- Working Load Limit of 250kg

## Specifications

EvacSHEET size: Standard - 1.90m x 0.85m 1.6kg Paediatric – 1.20m x 0.60m 1.3kg Trolley open – 1.65m x 0.80m Trolley packed – 0.30m x 0.20m x 0.10m 1.6kg Safety tested to: More than 350kg Working Load Limit (WLL): 250kg – Minimum 2 operatives safe working Material: Heavy duty (500D), Flame Retardant PU coated Polyester Buckles: 3 x 50mm, Black ACETAL, minimum tensile strength 100kg Pulling and Cross Straps: 48mm Polyester Webbing, 58 gsm / m, Break strength 2.6KN Country of Origin: Manufactured in United Kingdom. Conforming Standard: CA 015788 (MHRA)

## Laundering Instructions

Washing: When laundering becomes necessary, the buckles on the EvacSHEET should be fastened & the EvacSHEET can be washed in temperatures up to 70 °C for a disinfectant clean

Ensure the EvacSHEET has a thorough rinse to remove any detergent or disinfectant deposits. MACHINE WASHABLE, DO NOT BLEECH, DO NOT TUMBLE DRY, DRIP DRY / AIR DRY ONLY.

Additional information: Phone: 061 622 484 Email: info@astralhealthcare.com

